

How to Care for Your Fitted Table Cover

CLEAN your upholstered items when needed using a mild solution of 4:1 diluted non-alcohol cleaner such as Formula 409®, Fantastik®, or Green Windex®. The Fitted Massage Table Cover may be machine washed occasionally as needed using mild detergents and cold water, and tumble dried on low heat.

DISINFECT your fabric with any of the disinfectants listed below, following the Manufacturer's disinfections instructions.

Products listed below meet EPA's criteria for use against SARS-CoV-2, the virus that causes COVID-19

- Clorox Everest / Formula 409® Multi-Surface Cleaner (EPA Reg. No. 5813-73)
- Metrex Cavicide1 (EPA Reg. No. 46781-12)
- Parker Labs: Protex™1 (EPA Reg. No. 6836-152)
- Clorox 10% household (Bleach) Solution (EPA Reg. No. 5813-100)
- Clorox 8.25% Germicidal Bleach (EPA Reg. No. 67619-32)
- Clorox Healthcare 0.65% Bleach Germicidal Cleaner Spray (EPA Reg. No. 56392-7)
- Clorox 1.4% Hydrogen Peroxide Disinfectant (EPA Reg. No. 67619-24)
- Madacide-FD (EPA Reg. No. 1130-15)
- Clorox Disinfecting Wipes (EPA Reg. No. 67619-9)
- Clorox Healthcare VeraSure Wipes (EPA Reg. No. 67619-37)
- Clorox Healthcare Fuzion Cleaner Disinfectant (EPA Reg. No. 67619-30)

Additional Disinfectants


- 3M C. diff Solution Tablets (EPA Reg. No. 718147-6)
- AntiGone Wipes (SC 291919-10)
- Clorox EZ Kill Wipes (EPA Reg. No. 5984-10)

DO NOT use citrus based cleaners or other strong cleaners, such as alcohol, acetone, higher concentrations of bleach.

DO NOT expose the fabric to temperatures below 32°F/0°C or above 95°F/35°C.

DO NOT expose the fabric to direct sunlight, adhesives, liquids, or abrasive materials.

FAILURE TO FOLLOW RECOMMENDATIONS WILL VOID
THE WARRANTY ON THE FABRIC.

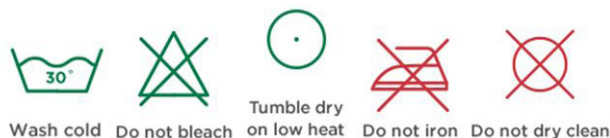

*If you have any questions, call Customer Service
at 717-235-6807 or visit our website:
www.oakworks.com*